

Detailed Instruction Guide

A Hollister Technology


Faecal Collectors

Application Guide


Preparing the skin

Carefully trim hair in the perianal area as close to the skin as possible. **Do not** use a razor. Cleanse the area using any mild soap and water. Rinse thoroughly and dry the skin completely before proceeding. Fill in irregular skin depressions in the perianal area, using a skin protective paste.


Applying the Drainable Faecal Collector

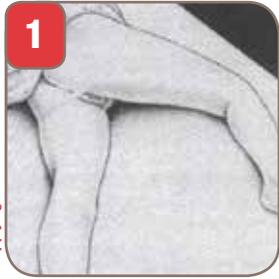

Have the patient assume a side-lying position with the upper knee toward the chest to expose the perianal area. The patient should remain in this position for the entire application procedure. Remove the collector and discard the plastic cover which protects the skin barrier. **Close the cap on the convenience drain.**

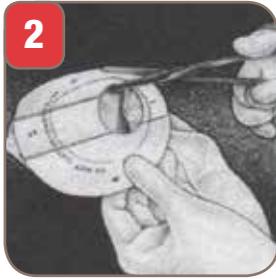

The opening in the skin barrier may be enlarged if necessary, but do not cut beyond the line indicated on the backing paper.

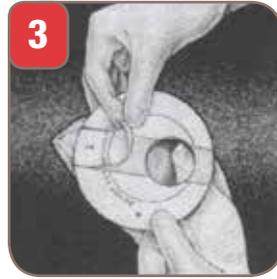

Remove the release paper from the sections of the skin barrier numbered 1 and 2.a



Fold the barrier in half and separate the patient's buttocks to expose the perianal area. For **female** patients adhere the narrowest section of the ring (numbered 1) to the skin between the anus and the vagina.




Press the barrier firmly against the skin to ensure a secure bond. For **male** patients adhere this section of the barrier (numbered 1) to the skin between the anus and the scrotum.



Place the upper portion of the exposed skin barrier (numbered 2) against the coccygeal area and press firmly against the skin. Release the buttocks, allowing them to assume a relaxed position.



Remove the release paper from the portion of the skin barrier nearest the patient's lower buttock (numbered 3 or 4, depending on the patient's position). Press the barrier firmly against the skin.

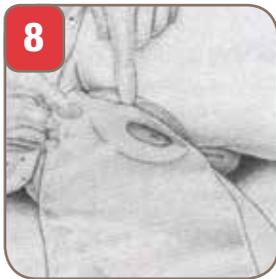

Remove the remaining piece of release paper (numbered 3 or 4) and place the skin barrier against the skin. **Be sure the barrier adheres firmly to the skin at all points.**



Draining the Collector

Have the patient assume a side-lying position to provide access to the collector. Place a bedpan or other receptacle under the convenience drain and remove the cap. When the collector is empty, wipe the end of the drain with tissue and replace cap. For continuous collection, the drainable faecal collector may be connected to a bedside drainage collector using standard tubing.



The collector features a gas vent near the upper end of the collector which allows gas to escape, keeping the collector from inflating.



Removing the Faecal Collector

Gently peel the skin barrier away from the patient's skin.



Seal the collector by folding skin barrier in half so it adheres to itself. Kindly follow your local infection control guidelines for correct disposal of the faecal collector. **Do not flush down the toilet.**



Your opportunity to see how simple and easy it is to fit.

Available on Drug Tariff

Faecal Collectors				
Drug Tariff-FP10 Code	PIP Code	NHS Product Code	Capacity (ml)	Box Quantity
9822	025-3989	GCT042	500 ml	10
9821	025-3791	GCT043	1000 ml	10