



CeraPlus™ Ostomy Product* Guide



Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches


Paediatric Pouchkins™

Skin Essentials

Resources

Welcome to your Hollister Ostomy Product Guide

Within this guide you will find a range of products designed to suit individual needs, including CeraPlus™ Products, an advanced line of ostomy products that help keep skin healthy.

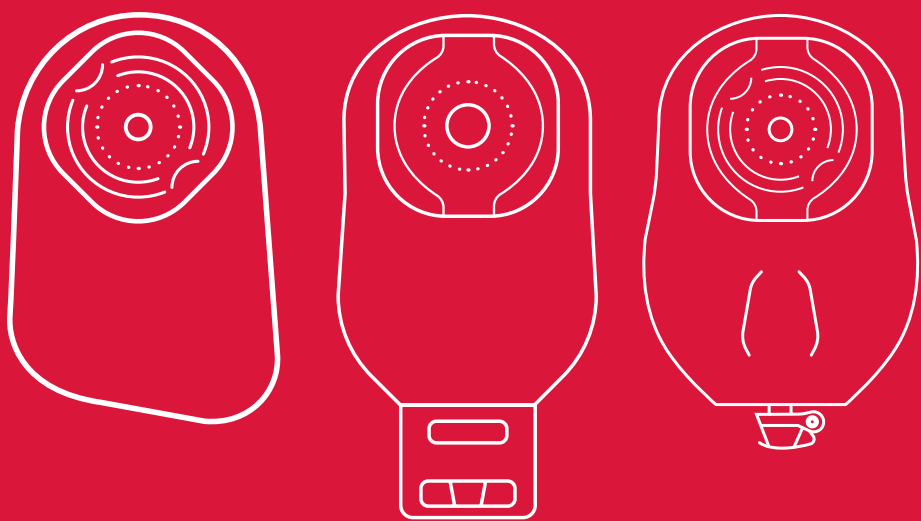

Hollister CeraPlus™ Ostomy Products have received official dermatological accreditation from the Skin Health Alliance, which may bring confidence and reassurance to end users.





Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

CeraPlus™ Ostomy Products

Protection Where it Matters Most

CeraPlus™ Products are demonstrated by evidence to reduce the occurrence and impact of PSCs



As many as **75% of ostomates** have peristomal skin complications (PSCs).¹ CeraPlus™ Products protect skin from Day 1 making it an ideal **first-line choice for your ostomy patients**. When compared to non-CeraPlus™ Skin Barriers, significantly more (75%) patients were “**very satisfied**” with the overall performance of CeraPlus™ Products.²

- Helps support and maintain healthy peristomal skin²
- Delivers positive outcomes for ostomy-related costs and care²
- Provides fit with convexity options and adhesion performance to enhance skin health²



Click here for
Closed Pouches

CeraPlus™ One-Piece Closed Pouches

Skin Health, Comfort and Security Combined

Designed with patients in mind
Designed for comfort, the Hollister Soft Convex pouching system is available in a range of sizes to suit individual needs*

Comfortable and secure fit
Designed to conform to individual and changeable body shapes for comfort and security against leakage

Built in belt loops
Option to attach a belt for peace of mind and additional pressure if required

Ceramide infused skin barrier
Ceramide is a natural component of human skin. It helps prevent water loss that can lead to skin damage and dryness²

Integrated soft convex gasket
Provides a secure and comfortable fit whilst applying gentle pressure to help protrude the stoma further into the bag

Soft Convexity may benefit people who:

- **Have a stoma hidden** behind skin folds or scars
- **Have a flush stoma**
- **Have a moat or skin crevice** surrounding the stoma
- **Are experiencing leakage** with their pouch



Click here for more **Closed Pouches**

CeraPlus™ One-Piece Closed Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX			
MIDI	Beige+	Clear	Pack size
15-38 mm	52200	-	30
15-55 mm	52201	-	30
MAXI	Beige+	Clear	Pack size
15-38 mm	522100	-	30
15-55 mm	522101	-	30
25 mm	522125	-	30
30 mm	522130	-	30
35 mm	522135	-	30
40 mm	522140	-	30



Click here for more **Closed Pouches**

CeraPlus™ One-Piece Closed Pouches



CeraPlus™ Convex Pouches


CONVEX			
MIDI	Beige+	Clear	Pack size
15-38 mm	52600	-	30
15-51 mm	52601	-	30
25 mm	52625	-	30
30 mm	52630	-	30
35 mm	52635	-	30

MAXI	Beige+	Clear	Pack size
15-38 mm	526100	-	30
15-51 mm	526101	-	30



Click here for more
Closed Pouches

CeraPlus™ One-Piece Closed Pouches

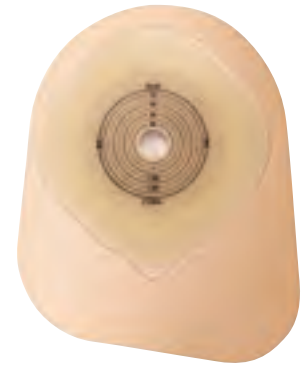

CeraPlus™ Flat Pouches

FLAT			
MIDI	Beige+	Clear	Pack size
15-55 mm	52100	-	30
25 mm	52125	-	30
30 mm	52130	-	30
35 mm	52135	-	30
MAXI	Beige+	Clear	Pack size
15-64 mm	521101	-	30

+Pouch features Viewing Option

When the flap is lifted, the stoma and output can be observed clearly and privately. This feature can also help patients position the pouch more easily, and help maintain their peace of mind.



Click here for **Drainable Pouches**

CeraPlus™ One-Piece Drainable Pouches

Skin Health, Comfort and Security Combined

Designed with patients in mind
Designed for comfort, the Hollister Soft Convex pouching system is available in a range of sizes to suit individual needs*

Comfortable and secure fit
Designed to conform to individual and changeable body shapes for comfort and security against leakage

Built in belt loops
Option to attach a belt for peace of mind and additional pressure if required

Ceramide infused skin barrier
Ceramide is a natural component of human skin. It helps prevent water loss that can lead to skin damage and dryness²

Integrated soft convex gasket
Provides a secure and comfortable fit whilst applying gentle pressure to help protrude the stoma further into the bag

Soft Convexity may benefit people who:

- **Have a stoma hidden** behind skin folds or scars
- **Have a flush stoma**
- **Have a moat or skin crevice** surrounding the stoma
- **Are experiencing leakage** with their pouch



Click here for more **Drainable Pouches**

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX			
MIDI	Beige+	Clear	Pack size
15-38 mm	540200	530200	30
15-55 mm	540201	530201	30
25 mm	540225	-	30
29 mm	540229	-	30
32 mm	540232	-	30
MAXI	Beige+	Clear	Pack size
15-38 mm	541200	531200	30
15-55 mm	541201	531201	30
25 mm	541225	-	30
29 mm	541229	-	30
32 mm	541232	-	30
35 mm	541235	-	30
38 mm	541238	-	30



Click here for more
Drainable Pouches

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX with Adhesive Border

MIDI	Beige+	Clear	Pack size
15-25 mm	54002	53002	30
15-38 mm	54000	53000	30
15-55 mm	54001	53001	30
25 mm	54025	-	30
30 mm	54030	-	30
35 mm	54035	-	30

MAXI	Beige+	Clear	Pack size
15-25 mm	54102	53102	30
15-38 mm	54100	53100	30
15-55 mm	54101	53101	30
25 mm	54125	-	30
30 mm	54130	-	30
35 mm	54135	-	30



Click here for more **Drainable Pouches**

Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Convex Pouches

CONVEX			
MIDI	Beige+	Clear	Pack size
15-38 mm	546200	-	30
15-51 mm	546201	-	30
MAXI	Beige+	Clear	Pack size
15-38 mm	548200	-	30
15-51 mm	548201	-	30



Click here for more
Drainable Pouches

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Convex Pouches

CONVEX with Adhesive Border

MIDI	Beige+	Clear	Pack size
15-25 mm	54602	53702	30
15-38 mm	54600	53700	30
15-51 mm	54601	53701	30
25 mm	54625	-	30
30 mm	54630	53730	30
35 mm	54635	53735	30

MAXI	Beige+	Clear	Pack size
15-25 mm	54802	53902	30
15-38 mm	54800	53900	30
15-51 mm	54801	53901	30
25 mm	54825	-	30
30 mm	54830	53930	30
35 mm	54835	53935	30



Click here for more
Drainable Pouches

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Flat Pouches

FLAT			
MIDI	Beige+	Clear	Pack size
15-64 mm	542200	-	30
MAXI	Beige+	Clear	Pack size
15-64 mm	543200	534200	30



Click here for more
Drainable Pouches

CeraPlus™ One-Piece Drainable Pouches



CeraPlus™ Flat Pouches

FLAT with Adhesive Border			
MIDI	Beige ⁺	Clear	Pack size
13-64 mm	54200	53500	30
25 mm	54225	53525	30
30 mm	54230	53530	30
35 mm	54235	53535	30
MAXI	Beige ⁺	Clear	Pack size
13-64 mm	54300	53400	30
25 mm	54325	53425	30
30 mm	54330	53430	30
35 mm	54335	53435	30



Click here for more
Drainable Pouches Information

CeraPlus™ One-Piece Drainable Pouches



Flextend™ Skin Barrier

High Output with Soft Drain Tap

FLAT with Adhesive Border

MAXI	Beige	Clear	Pack size
13-64 mm	-	26064	10

High Output Accessories

	Order No.	Pack size
High Output Universal Adaptor	7332	10
High Output Collection Bag (Drainable 2000mL)	5551*	10

*Only available to order through NHS supply chain on GCZ85007



Lock 'n Roll™

The microseal closure contains over a thousand tiny microseals which securely engage when pressed together, which should create a water, weight, and pressure-resistant seal.



CeraPlus™ One-Piece Urostomy Pouches

Skin Health, Comfort and Security Combined

Designed with patients in mind

Designed for comfort, the Hollister Soft Convex pouching system is available in a range of sizes to suit individual needs*



Built in belt loops

Option to attach a belt for peace of mind and additional pressure if required

Enhanced flow-control tap

The tap is made with softer material and a streamlined shape. The goal of this design? To help increase comfort against the body, so your patients can do more, with less worry.



Multi-chamber design

The pouches have multiple built-in chambers, which are designed to facilitate even distribution of urine. As urine flows into the pouch, it is dispersed among the chambers. This results in a more balanced collection of urine, and a lower pouch profile for your patients.



Comfortable and secure fit

Designed to conform to individual and changeable body shapes for comfort and security against leakage.



Ceramide infused skin barrier

Ceramide is a natural component of human skin. It helps prevent water loss that can lead to skin damage and dryness²



Integrated soft convex gasket

Provides a secure and comfortable fit whilst applying gentle pressure to help the stoma protrude further into the bag.

Soft Convexity may benefit people who:

- **Have a stoma hidden** behind skin folds or scars
- **Have a flush stoma**
- **Have a moat or skin crevice** surrounding the stoma
- **Are experiencing leakage** with their pouch



Click here for more **Urostomy Pouches**

CeraPlus™ One-Piece Urostomy Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX

	Beige+	Clear	Pack size
15-25 mm	598200	599200	10
15-38 mm	598201	599201	10
20 mm	-	599220	10
25 mm	-	599225	10
30 mm	-	599230	10



Click here for more **Urostomy Pouches**

CeraPlus™ One-Piece Urostomy Pouches



CeraPlus™ Soft Convex Pouches

SOFT CONVEX with Adhesive Border

	Beige+	Clear	Pack size
13-25 mm	-	59900	10
13-38 mm	-	59901	10
13-55 mm	-	59902	10
25 mm	59825	-	10
30 mm	59830	-	10



Click here for more **Urostomy Pouches**

CeraPlus™ One-Piece Urostomy Pouches



CeraPlus™ Convex Pouches

CONVEX			
	Beige+	Clear	Pack size
15-25 mm	596200	597200	10
15-38 mm	596201	597201	10



CeraPlus™ Convex Pouches

CONVEX with Adhesive Border			
	Beige	Clear	Pack size
13-25 mm	-	59700	10
13-38 mm	-	59701	10
13-51 mm	-	59702	10



Click here for more Urostomy Pouches

CeraPlus™ One-Piece Urostomy Pouches



CeraPlus™ Flat Pouches

FLAT with Adhesive Border

	Beige	Clear	Pack size
13-64 mm	-	59400	10

Urostomy Essentials

	Order No.	Pack size
Urostomy Night Drainage Bag (2000mL)	5550	10
Urostomy Drain Tube Adapter	7331	10

Urinary Leg Bags

Sterile Leg Bag 500mL 10 cm inlet (regular)	9621	10
Sterile Leg Bag 500mL variable inlet (regular)	9624	10
Sterile Leg Bag 800mL 10 cm inlet (regular)	9631	10
Sterile Leg Bag 800mL 50 cm variable inlet (regular)	9632	10



Click here for
Two-Piece Skin Barriers




CeraPlus™ Two-Piece Skin Barriers



CeraPlus™ Skin Barrier

SOFT CONVEX with Adhesive Border

Stoma	Gasket	Order No.	Pack size
13-25 mm	45 mm	14A00	5
13-38 mm	55 mm	15A00	5
13-51 mm	70 mm	17A00	5
22 mm	45 mm	14A20	5
25 mm	45 mm	14A25	5
29 mm	55 mm	15A29	5
32 mm	55 mm	15A32	5

-  45 mm Gasket
-  55 mm Gasket
-  70 mm Gasket



Click here for more
Two-Piece Skin Barriers




CeraPlus™ Two-Piece Skin Barriers



CeraPlus™ Skin Barrier

CONVEX with Adhesive Border


Stoma	Gasket	Order No.	Pack size
13-25 mm	45 mm	14600	5
13-38 mm	55 mm	15600	5
13-51 mm	70 mm	17600	5
20 mm	45 mm	14620	5
25 mm	45 mm	14625	5
29 mm	55 mm	15629	5
32 mm	55 mm	15632	5

-  45 mm Gasket
-  55 mm Gasket
-  70 mm Gasket



Click here for more
Two-Piece Skin Barriers

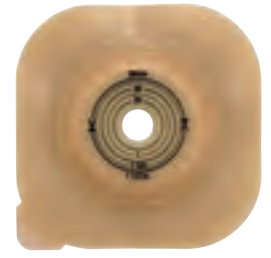


CeraPlus™ Two-Piece Skin Barriers



CeraPlus™ Skin Barrier

CONVEX

Stoma	Gasket	Order No.	Pack size
13-25 mm	45 mm	14100	5
13-38 mm	55 mm	15100	5
13-51 mm	70 mm	17100	5

-  45 mm Gasket
-  55 mm Gasket
-  70 mm Gasket



Click here for more
Two-Piece Skin Barriers

CeraPlus™ Two-Piece Skin Barriers



CeraPlus™ Skin Barrier

FLAT with Adhesive Border

Stoma	Gasket	Order No.	Pack size
13-32 mm	45 mm	14500	5
13-44 mm	55 mm	15500	5
13-57 mm	70 mm	17500	5



CeraPlus™ Skin Barrier

FLAT

Stoma	Gasket	Order No.	Pack size
13-32 mm	45 mm	14200	5
13-44 mm	55 mm	15200	5
13-57 mm	70 mm	17200	5



Click here for **Two-Piece Conform 2 Pouches**

Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

Two-Piece Conform 2 Pouches



Closed Pouch

CLOSED Pouch			
MINI	Beige	Clear	Pack size
45 mm	24410	-	30
55 mm	25410	-	30
MIDI	Beige	Clear	Pack size
45 mm	24400	24500	30
55 mm	25400	25500	30
70 mm	27400	27500	30
MAXI	Beige	Clear	Pack size
45 mm	24420	24520	30
55 mm	25420	25520	30
70 mm	27420	27520	30

- 45 mm Gasket
- 55 mm Gasket
- 70 mm Gasket



Click here for more
Two-Piece Conform 2 Pouches

Two-Piece Conform 2 Pouches



Drainable Pouch

DRAINABLE Pouch

MINI	Beige	Clear	Pack size
45 mm	24743	-	30
55 mm	25743	-	30

MIDI	Beige	Clear	Pack size
45 mm	24750	-	-
55 mm	25750	25850	30

MAXI	Beige	Clear	Pack size
45 mm	24760	24860	30
55 mm	25760	25860	30
70 mm	27760	27860	30

MINI Pouch*

MIDI	Beige	Clear	Pack size
45 mm	24720	-	30
55 mm	25720	-	30

*Requires a clip



Click here for more
Two-Piece Conform 2 Pouches

Two-Piece Conform 2 Pouches



Urostomy Pouch

UROSTOMY Pouch with Soft Tap

MIDI	Beige	Clear	Pack size
45 mm	24790	24890	10
55 mm	25790	25890	10
70 mm	-	27890	10

- 45 mm Gasket
- 55 mm Gasket
- 70 mm Gasket



Click here for
Paediatric Pouchkins

Paediatric Pouchkins™



Premature Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>Ultra clear one-piece pouch with barrier CTF to 15 mm</p> <ul style="list-style-type: none"> • 50 mm round barrier footprint • No starter hole • With soft wire closure if required 	50	No	3777	15



Newborn Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>Ultra clear one-piece pouch with barrier CTF to 35 mm x 22 mm</p> <ul style="list-style-type: none"> • 64 mm round barrier footprint • No starter hole • With soft wire closure if required 	190	No	3778	15



Click here for more Paediatric Pouchkins

Paediatric Pouchkins™



Child Drainable Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>One-piece clear drainable pouch with Lock'n Roll™ CTF to 50 mm</p> <ul style="list-style-type: none"> • 100 mm triangular barrier footprint • No starter hole 	280	Yes	3795	10



Infant Drainable Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>One-piece clear drainable pouch with Lock'n Roll™ CTF to 38 mm</p> <ul style="list-style-type: none"> • 70 mm square barrier footprint • No starter hole 	230	No	3796	10



Click here for more Paediatric Pouchkins

Paediatric Pouchkins™



Infant Urostomy Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>One-piece clear urostomy pouch CTF to 38 mm</p> <ul style="list-style-type: none"> • 70 mm square barrier footprint • No starter hole 	230	No	3797	10



Infant Drainable Pouch

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>Two-piece clear drainable pouch with Lock'n Roll™ closure</p> <ul style="list-style-type: none"> • 45 mm flange 	230	No	3799	10



Click here for more Paediatric Pouchkins

Paediatric Pouchkins™



Infant Floating Flange

	ISO Vol. (ml)	Filter	Order No.	Pack size
<p>Two-piece barrier flange 45 mm CTF to 32 mm</p> <ul style="list-style-type: none"> • 85 mm square barrier footprint • No starter hole 	n/a	n/a	3761	5



SoftFlex™ Skin Barrier Drainable Pouch

MINI	Beige	Pack size
15-55 mm	28100	30
25 mm	28125	30
30 mm	28130	30
35 mm	28135	30



Click here for **Barrier Rings**

Skin Essentials CeraRing™ Barrier Rings

The confidence of a secure seal to help prevent leakage

- Infused with ceramide to protect your skin
- Fills dips and creases around your stoma, for a smooth surface to improve barrier fit
- Stretch, mould or stack to customise with confidence
- Alcohol free, no sting



[Click here](#) for more info or to order a sample



[Click here](#) for more **Barrier Rings**

Skin Essentials CeraRing™ Barrier Rings



CeraRing™ Barrier Rings

	Order No.	Pack size
CeraRing™ Barrier Ring 18 mm	8805	10
Slim CeraRing™ Barrier Ring 18 mm	8815	10
Slim CeraRing™ Barrier Ring 27 mm	881527	10



CeraRing™ Convex Barrier Rings

CeraRing™ Convex Barrier Ring 20 mm	89520	10
CeraRing™ Convex Barrier Ring 30 mm	89530	10
CeraRing™ Convex Barrier Ring 40 mm	89540	10
CeraRing™ Oval Convex Barrier Ring 22-38 mm	89601	10
CeraRing™ Oval Convex Barrier Ring 30-48 mm	89602	10
CeraRing™ Oval Convex Barrier Ring 38-56 mm	89603	10



Click here for more
info or to order a sample



Click here for more
Barrier Rings

Using CeraRing™ Barrier Rings

Option 1:

Apply flat or convex CeraRing™ Barrier Ring to the adhesive side of your skin barrier.



Stretch and shape the barrier ring for a customised fit.

Option 2:

Apply flat or convex CeraRing™ Barrier Ring to your skin, then apply your pouching system. Make sure the barrier ring fits where the skin and stoma meet.



 [Click here](#) for more info or to order a sample

 [Click here for Barrier Extenders](#)

Skin Essentials CeraPlus™ Barrier Extenders

The confidence of enhancing your pouch security

Enhance the feeling of pouch security by adding a frame to your skin barrier, also helping to extend pouch wear-time.




Increase the feeling of security when undertaking physical activity



Suitable for use in warm climates to reduce barrier edge lifting

Using CeraPlus™ Barrier Extenders

1. Remove existing skin barrier and clean the skin around the stoma. Apply new skin barrier.
2. Remove paper backing by stretching lightly at each end of the barrier extender.
3. Release only one side of the paper backing.
4. Place the adhesive end of the barrier extender along the perimeter of the skin barrier; covering both the skin barrier and the skin next to it. Remove remaining paper backing.
5. Repeat for any additional strips you require around your barrier.
6. Barrier extenders will remove easily with your routine pouch change.

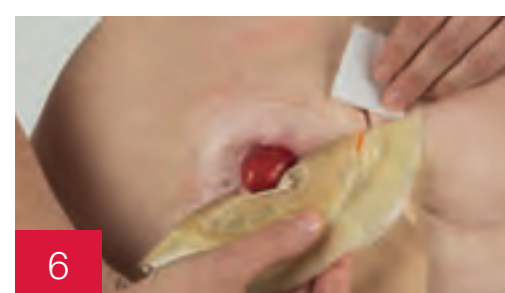

Click here for more info or to order a sample



Click here for more **Barrier Extenders**

Skin Essentials CeraPlus™ Barrier Extenders



Barrier Extenders

	Order No.	Pack size
CeraPlus™ Barrier Extenders (3 sections of hydrocolloid material)	79402	30



Click here for more info or to order a sample



Click here for **Spray and Wipes**

Skin Essentials Adapt™ Remover Spray and Wipes

The confidence of an easy pouch removal
that is gentle on skin

Using Adhesive Remover Spray

1. Shake well before use.
2. Peel back one edge of skin barrier.
3. Hold the can so arrow faces the base of the skin barrier, and spray until the adhesive releases.
4. Ensure skin is dry and apply new pouching system as usual.

Spray

Spray until the skin barrier releases.



Using Adhesive Remover Wipes

1. Press or rub the wipe against your skin as you peel off your skin barrier and/or tape.
2. Press or rub the wipe over any areas of the skin where adhesive residue remains.
3. Ensure the skin is clean and dry and apply new ostomy pouching system as usual.

Wipes

Rub the wipe over skin as you peel off skin barrier.



Wipes

Clean off remaining adhesive residue.



- Gently removes adhesive products and residue to make removal of skin barriers fast and easy
- Help reduce potential for skin stripping



[Click here](#) for more info or to order a sample



[Click here](#) for more **Spray and Wipes**

Closed
Pouches

Drainable
Pouches

Urostomy
Pouches

Two-Piece
Pouches

Paediatric
Pouchkins™

Skin Essentials

Resources

Skin Essentials Adapt™ Remover Spray and Wipes



Adhesive Remover Spray and Remover Wipes

	Order No.	Pack size
Adapt™ No Sting Medical Adhesive Remover Aerosol (50mL)	7737	1
Adapt™ No Sting Medical Adhesive Remover Aerosol (100mL)	7731	1
Universal Removal Wipes	7760	50



Click here for more info or to order a sample



Click here for **Lubricating Deodorant**

Skin Essentials Adapt™ Lubricating Deodorant

The confidence of eliminating odour
and easy pouch emptying

Using in Drainable Pouches

1. Apply pouch.
2. Bottle: add approximately 5ml or 1tsp (Figure 1).
Sachet: add the complete contents of the packet (Figure 2).
Close the pouch in the normal manner.
3. Rub to thoroughly coat the inside of the pouch (Figure 3).
Repeat steps 1 or 2 after every emptying.

Using in Closed Pouches

1. Do not remove release paper on barrier to add product.
2. Bottle: add approximately 5ml or 1tsp (Figure 4).
Sachet: add the complete contents of the packet.
3. Apply your pouch. Rub to thoroughly coat the inside of the pouch (Figure 3).



Lubricate the inside of the pouch to help limit pancaking
and for easy emptying.



[Click here](#) for more
info or to order a sample



[Click here](#) for more
Lubricating Deodorant

Skin Essentials Adapt™ Lubricating Deodorant



Deodorant

	Order No.	Pack size
Adapt™ Lubricating Deodorant Bottle (236mL)	78500	1
Adapt™ Lubricating Deodorant Sachets (8mL each)	78501	50



[Click here](#) for more info or to order a sample



[Click here](#) for **Skin Essentials**

Skin Essentials



Belts

	Order No.	Pack size
Adapt™ Belt Small/Medium (58-109 cm)	7300	1
Adapt™ Belt Large (74-124 cm)	7299	1



Powder

Adapt™ Stoma Powder (28g)	7906	1
---------------------------	------	---



Paste

Adapt™ Paste (60g)	79300	1
Adapt™ Paste (20x14g)	79301	1



Click here for more
Skin Essentials

Skin Essentials



Leg Bag Holsters

	Order No.	Pack size
Leg Bag Holsters (small)	9612	4
Leg Bag Holsters (medium)	9613	4
Leg Bag Holsters (large)	9614	4



Night Drainage Bags 2000mL

Night Drainage Bag (Non-Drainable)	9651	10
Night Drainage Bag (Drainable)	9650	10



Click here for more
Skin Essentials

Skin Essentials

Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

High Output Collection Bag 2000mL

	Order No.	Pack size
High Output Collection Bag (drainable)	5551*	10
High Output Universal Adaptor	7332	10

*Only available to order through NHS supply chain on GCZ85007



Urinary Leg Bags

Sterile Leg Bag 500mL 10 cm inlet (regular)	9621	10
Sterile Leg Bag 500mL variable inlet (regular)	9624	10
Sterile Leg Bag 800mL 10 cm inlet (regular)	9631	10
Sterile Leg Bag 800mL 50 cm variable inlet (regular)	9632	10



Click here for more **Skin Essentials**

Skin Essentials



Urostomy Essentials

	Order No.	Pack size
Urostomy Night Drainage Bag (2000mL)	5550	10
Urostomy Drain Tube Adapter	7331	10



Leg Bags and Night Drainage Bags Accessories

Leg Bag Straps	9625	10
Deluxe Leg Bag Straps (35 cm extra small)	9663	2
Deluxe Leg Bag Straps (38 cm small)	9660	2
Deluxe Leg Bag Straps (49 cm medium)	9661	2
Deluxe Leg Bag Straps (56 cm large)	9662	2



Click here for more
Skin Essentials

Skin Essentials



Barrier Rings

	Order No.	Pack size
Adapt™ Barrier Ring Flextend M™ (98 mm)	7806	10



Miscellaneous

CeraPlus™ 10 cm x 10 cm Skin Barrier	8900	5
Skin Gel Wipes	7917	50
Universal Catheter Access Port (Sterile)	9779	8
Adapt™ Barrier Strips (6g each) Flextend M™	79400	10




Click here for
Resources

Resources

We are here to help with information and education so everyone can live their best life with an ostomy.

Click on one of the booklets to view or download

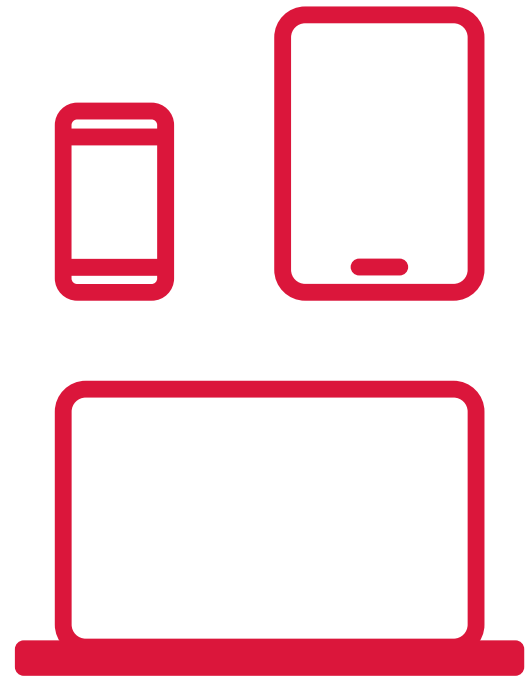

Understanding your Colostomy



Caring for Your Loved One with a Stoma



Understanding your Ileostomy



Understanding your Urostomy



Click here for more Resources

Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

Resources



Caring for Your Child with a Stoma



Living with a Stoma: Travel



Living with a Stoma: Healthy Eating



Living with a Stoma: Sex & Parenthood



Living with a Stoma: Home & Work Life



Living with a Stoma: Sports & Fitness



Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

Secure StartSM

Stoma Support Service

The **Secure StartSM** service works in partnership with you in the hospital. We are a team of advisors who have been trained by nurses, and we specialise in answering any day to day, or routine questions your patients may have about stoma care. We work alongside you to offer you support and guidance with any stoma related query.

99%

of patients found the support useful

“Lovely people checking up on me, made me feel very cared for and so nice to have this sort of aftercare.”

Patient

The **Secure StartSM** service provides support for patients who have recently undergone stoma forming surgery.

Welcome to **Secure StartSM** Services

All **Secure StartSM** registered patients can use this service, along with their relatives or carers (with permission). Confidential issues will only be discussed with the patient.

We are able to support your patients if they experience:

- Leakage
- Bag will not stay on
- Sore skin
- Change in length or size of stoma
- Stoma stops working
- Stoma becomes black
- Watery / loose output
- Feeling generally unwell, including nausea





Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources


“Patients receive a high quality service which impacts positively on their rehabilitation. The readmission of patients has been prevented by this level of community support.”

Lead clinical nurse specialist colorectal stoma care

Secure StartSM advisors

0800 376 1310

Mon - Fri
8.30am - 5pm

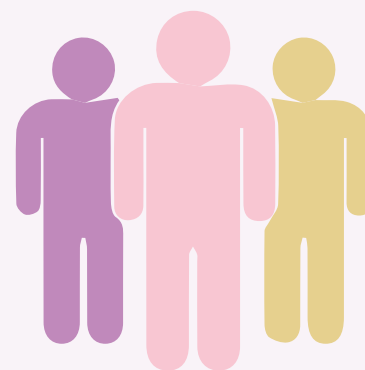

Secure StartSM weekend on-call nurse line

0800 592 202

Sat - Sun & Bank Holidays
9.00am - 1.30pm

In addition, patients can access some great advice, education and general stoma related information, by visiting our website:
www.securestart.co.uk





Compassionate Care, Confident Living.

However the journey begins, we're by your side with proactive, dependable care for your patients. Ensuring their needs are met with compassion, so they can live their life with confidence.



Trusted

With over 40 years' experience, you can rely on our dedicated team to continue the support and care as patients adopt a new way of life. We are Hollister UK's dispenser of choice and work closely with specialist nurses and GPs, fulfilling more than 50,000 NHS prescription orders each month.



Dependable

Our thoughtful, compassionate and personal service provides everything your patients need, directly and discreetly at their door.



Personal

Together, we work as a member of your team to help your patients live a dignified and confident life. We are available to provide discreet support to your patients and answer any questions, always ensuring a personalised service.



Closed Pouches

Drainable Pouches

Urostomy Pouches

Two-Piece Pouches

Paediatric Pouchkins™

Skin Essentials

Resources

Complimentary Items

A variety of complimentary items are available with patient orders, including dry wipes, wet wipes, and black disposal bags. A Fittleworth wash bag is supplied to every new patient registered.



Support Resources

We know that living with a condition brings challenges and we want to assure you that your patients are not alone in facing them. By working closely with clients, clinical nurses and a variety of qualified experts we've created a suite of FREE highly effective support materials, to help your patients live a dignified and confident life.

Visit our website: www.fittleworth.com and click the Services' tab and 'Free Resources' link for further information.

Registering your patient with us is as easy as 1,2,3



Call us

England & Wales: 0800 378 413 (HCP Team)
Scotland & N. Ireland: 0800 783 7148 (Option 2: HCP Team)
Mon to Fri 8am-6pm




Email us

England & Wales: fml.customerservices@nhs.net
Scotland & N. Ireland: fittleworthsc@nhs.net



Click here to Register online

Register your patient your way!

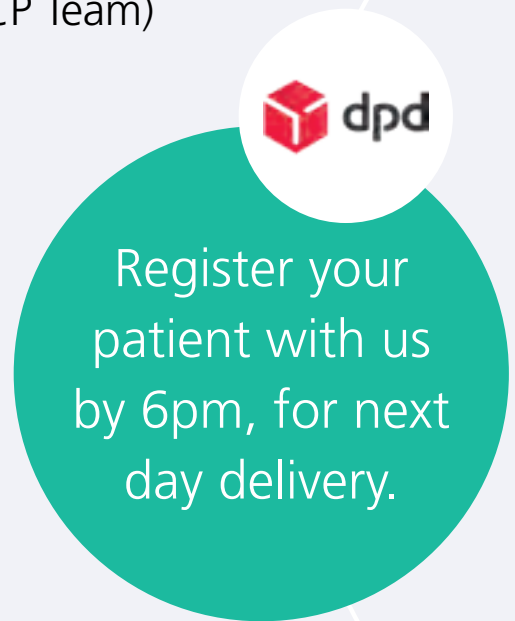

Hollister and the Hollister logo are trademarks of Hollister Incorporated.
© 2025 All Rights Reserved. Fittleworth Medical Limited.





Hollister CeraPlus™ Ostomy Products have received official dermatological accreditation from the Skin Health Alliance, which may bring confidence and reassurance to end users.

“CeraPlus Ostomy Products put my skin first, so I can focus on living my life without any concerns or disruptions.”

Ross, CeraPlus™ Product User



Closed
Pouches

Drainable
Pouches

Urostomy
Pouches

Two-Piece
Pouches

Paediatric
Pouchkins™

Skin Essentials

Resources

For more information or for **FREE** samples, contact us on:

Tel: 0800 521 377

email: samples.uk@hollister.com

To find out more about the best ways to maintain healthy skin, visit www.hollister.co.uk

UK

Hollister Limited

Building 1010

Winnersh Triangle Business Park

Eskdale Road

Winnersh

Wokingham

RG41 5TS

www.hollister.co.uk

References:

1. Smith AJ, Lyon CC, Hart CA. Multidisciplinary care of skin problems in stoma patients. Br J Nurs. 2002 Mar;11(5):324-30.
2. Colwell J, Pittman J, Raizman R, Salvadalena G. A Randomized Controlled Trial Determining Variances in Ostomy Skin Conditions (ADVOCATE). J Wound Ostomy Continence Nurs. 2018;45(1):37-42.

*CeraPlus™ Skin Barriers contain the Remois Technology of Alcare Co., Ltd.

Prior to the use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions. Not all products are CE marked. All other trademarks and copyrights are the property of their respective owners. Person received compensation from Hollister Incorporated. Hollister, the Hollister logo, Adapt, CeraPlus, CeraRing, Flexextend M, Lock 'n Roll, Pouchkins and Soft Flex are trademarks of Hollister Incorporated.

© 2025 Hollister Incorporated.



CeraPlus™
Ostomy Products

Closed
Pouches

Drainable
Pouches

Urostomy
Pouches

Two-Piece
Pouches

Paediatric
Pouchkins™

Skin Essentials

Resources